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## EN

### General Safety Instructions and Warnings “Care mounts”

#### 1. General

Dear Customer. Thank you for choosing a product from Rehadapt! Please read the following safety instructions and comply with all notices on the product in order to ensure optimum operation of the product. Please store the General Safety Instructions and Warnings carefully for future use. Should you pass on the product to a third party, please also pass on the accompanying documents for the product and the instructions for use. Rehadapt manufactures mounting for assistive devices that are designed – among others – for use with communication aids for persons with communicative restrictions due to disabilities.

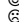
**1.1 Warning symbols:** The following symbols are used to indicate various safety and warning levels:

 Sign for the user: Particular attention is required here!

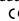
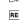
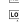
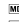
 Indication of potential damage or malfunctions of the product in the event of non-compliance.

 Warning of significant risk of damage for the user, the Rehadapt product or the mounted device in the event of non-compliance with the warning sign.

 Indication of significant risk of injury for the user in the event of non-compliance with the warning sign.

-  Prohibition sign. Non-compliance with such a sign will render the warranty void.
-  Correct or proper position / use
-  Impermissible or incorrect position / use
-  Acceptable or adequate position / use
-  Reference to additional attachments with separate instructions for use
-  Opening / loosening
-  Closing / tightening
-  Half opening or closing / slight loosening or closing

**1.2 Symbols and markings:** Meaning of symbols that can be found on Rehadapt products, accessories, in instructions for use or on the packaging:

-  CE is an abbreviation for European Community and this marking provides customs authorities in the European Union with the information that the product complies with one or more EC Directive(s).
-  The UKCA marking is the product marking that indicates compliance with the applicable requirements for products placed on the market in Great Britain (England, Scotland and Wales).
-  Do not re-use.
-  Comply with the instructions for use.
-  Article number
-  Batch designation
-  Protect against moisture.
-  Date of manufacture
-  Manufacturer
-  Serial number
-  Medical device
-  Unique Device Identifier
-  Chocking hazard. Small parts

### 1.3 Intended purpose

#### Intended use:

Rehadapt Care Mounts are modular mechanical systems for positioning, attaching, and organizing care aids. They are particularly suitable for the safe provision of medical electrical devices in home and institutional care settings.

Rehadapt Care Mounts are neither life-sustaining nor life-supporting (Risk Class I), are considered accessories to medical devices, and can be combined with medical devices up to Risk Class IIb according to their intended use. This applies to the following devices, but is not limited to:

- Ventilators
- Humidifiers for respiratory gases and sterile water
- Monitoring devices for respiratory gases
- Suction devices
- Cough assistants
- Pulse oximeters and oximeters
- Pumps for enteral nutrition and tube feeding
- Patient monitors

Rehadapt Care Mounts are specifically designed for use with medical electrical devices and support their function through:

- Positioning the devices for improved function (e.g., positioning relative to the patient, optimal distance between ventilation and humidification)
  - Positioning the devices for improved usability (e.g., visibility and clarity of all displays even at night and in emergency situations, access to controls and connections)
  - Optimized attachment of devices for reduced mechanical hazards (e.g., attachment for instability due to movement, strain relief to prevent devices from tipping or falling)
  - Attaching various devices directly to the care bed/cart for improved mobility within the room (e.g., moving the bed and/or cart for patient access or emergency assistance)
  - Integrated attachment of sterile water for humidifiers for respiratory gases and/or tube feeding for enteral nutrition to the care bed/cart for improved mobility within the room (e.g., provision close to the patient and easy mobility)
  - Positioning the ventilation tubing system for improved connection to the patient (e.g., routing behind the head for easy repositioning of the patient)
  - Attaching a ventilation tubing system for reduced strain (e.g., attachment to a movable headboard prevents interruption of gas supply, slipping, or pinching)
  - Fixed routing of the ventilation tubing system for reduced strain on the tracheostoma (e.g., reduction of irritation, inflammation, and adhesions)
  - Keeping backup devices ready for improved operational reliability and easy replacement (e.g., defined storage location for a second ventilator)
- Additionally, Rehadapt Care Mounts enable standardization of care processes through positioning, attachment, and organization, significantly expanding the available workspace, which leads to reduced physical and mental strain on caregivers. At the same time, risks in the care environment (e.g., fewer tripping hazards) can be reduced, and basic hygiene (e.g., easy room cleaning) can be improved.
- A Rehadapt Care Mount consists of at least one module from the following categories:
- Base Module: Attachment element that is semi-permanently attached to a stable element of a third-party product and forms the basis for a Rehadapt Care Mount.
  - Core Module: Combination of elements (tubes and connectors) that connect one or more base modules with one or more holder modules stably and/or movably, defining the configuration of the Rehadapt Care Mount.
  - Equipment Module: Attachment element, possibly with a quick-change system, that is semi-permanently attached to a third-party product or on which a third-party product is placed/stored and through which it is connected to the main module. The attachment and assembly of the Rehadapt Care Mount are done by screwing and/or clamping. The combination with care aids is done by screwing, inserting,

hanging, placing, or plugging. Rehadapt Care Mounts are supplemented with accessories as needed.

Rehadapt Care Mounts can be used indoors in application areas 3, 4, 5. Temporary outdoor use is possible. All relevant technical specifications (including compatible devices and system limits) are included in the accompanying documentation. The use of Rehadapt Care Mounts does not define the type or scope of use of medical electrical devices nor their exact positioning. The operator of the medical devices is responsible for ensuring that the combination of devices meets the requirements of EN 60601-1. The clinical condition of the patient requires separate care.

**Indication(s):** Rehadapt Care Mounts improve care, especially when multiple medical electrical devices are used and when patients are highly dependent on support. This applies to the following indications, but is not limited to:

- **Indication A1:** Care with at least one medical electrical device and medical self-care excluded.
- **Indication A2:** Care with at least one medical electrical device and medical self-care largely excluded, particularly due to lack of motor skills. Additionally, Rehadapt Care Mounts can improve care when medical electrical devices are used to a limited extent or when patients are not dependent on support. This applies to the following indications, but is not limited to:
- **Indication B:** Care with at least one medical electrical device and motor skills not significantly impaired.
- **Indication C:** Care without medical electrical devices, particularly for the provision of non-technical care aids.

**Contraindication(s):** none

**Patient target group:** Improvements in care are achieved for patient groups who are provided with multiple medical electrical devices and are dependent on support for their medical self-care. This applies to the following diagnoses, but is not limited to:

- Persistent vegetative state / Apallic syndrome
- Amyotrophic lateral sclerosis (ALS)
- Acute and chronic respiratory insufficiency
- Stroke / Apoplexy
- Brain hemorrhage / Intracerebral hemorrhage
- Muscular dystrophies
- Infantile cerebral palsy

Additionally, improvements in care can also be achieved for patient groups who are provided with few/no medical electrical devices or who are not dependent on support for their medical self-care. This applies to the following diagnoses, but is not limited to:

- Chronic obstructive pulmonary disease (COPD)
- Sleep apnea syndrome
- People with physical disabilities

**Intended users:**

Rehadapt Care Mounts are intended for installation and maintenance by qualified personnel of authorized dealers. They possess specific expertise. Operation is carried out by laypersons in care, as well as care personnel in home and institutional care. They do not possess product-specific expertise. Operation by the patient is not excluded if they have the appropriate motor skills.

**1.4 Scope of delivery:** Please compare the scope of delivery with the instructions for use of the respective product. Other supplementary products might be necessary, in order to benefit from full use of the product. These should be included in the delivery from your supplier. Should you have any questions in this regard, please contact your supplier.

## 2. Safety

Rehadapt products have been tested and comply with all specifications and standards for medical products. However, to ensure safe operation of the Rehadapt products, you should comply with the following safety instructions and warnings:

### 2.1 General Safety Instructions and Warnings

- ▲ Use of unauthorised accessories or unauthorised changes to the product. Unauthorised accessories or any unauthorised changes made to the product can result in danger and/or injury to persons and to material damage.
- Only ever use accessories approved by Rehadapt for using in combination with the product.
- Only make changes to the product if these have been approved by Rehadapt.
- ▲ Make sure the individual components are assembled correctly. Incorrect assembly can impair the stability and operation of the product and can cause damage to the mounted device and as a consequence result in physical injury.
- ▲ Small parts included in the scope of the delivery and packaging must be stored out of the reach of children. Risk of suffocation.
- ▲ Comply with the max. permissible load capacity of the product and the care bed/cart, in order to prevent damage to the Rehadapt product or the care bed/cart
- ▲ Never use force or excessive strength. This could damage the Rehadapt product or the mounted device.
- ▲ Risk of pinching. When using Rehadapt products, always ensure that body parts are kept away from moving parts, namely leave a 25 mm (1") gap between moving parts in accordance with DIN 60601-2-38.
- ▲ Rehadapt products are delivered ready for use. The use of additional grease or solvents is not necessary and may only be carried out by trained medical product advisers. See chapter "Maintenance and Service"
- ▲ Remove all attached devices before transporting the care bed/cart or removing bolts and levers, in order to prevent these from dropping and being damaged.
- ▲ Do not make modifications or adjustments to Care Mounts in close proximity to patients, and ensure all connected devices are removed beforehand.
- ▲ The care bed/cart with Care Mount should only be used on flat and even surfaces.

- ▲ The care bed/cart with Care Mount may only be moved within the care area and must be guided in a controlled manner during transport.
- ▲ Care Mounts are intended for indoor use only and must not be used outdoors on a permanent basis.
- ▲ The ends of the Care Mounts may extend beyond the care bed/cart and could have sharp edges. Always maintain a safe distance when walking around the Care Mount and any attached devices to avoid tripping or injury.
- ▲ Tighten the bolt connections or levers should the mount loosen.
- ! Do not attach any decorations or the like permanently to the Rehadapt product that are not included in the scope of delivery and scope of use, as these could impair the operation and the stability of the product. Make sure that any safety instructions or warnings on the product are clearly visible.

### 2.2 Assembly

- ▲ The user and the specialist staff of the supplier are responsible for safety and operation. Make sure the product is assembled correctly and firmly, to ensure reliable operation. Rehadapt excludes any liability for improper assembly.

### 2.3 Service life

- ▲ Using the device after expiry of its specified service life leads to an increase in the residual risks and should only be undertaken after careful consideration by qualified specialist staff of the supplier. Once the service life has expired, the user or a responsible person should contact the supplier. The supplier can then decide whether the product can be reconditioned or its use extended.

### 2.4 Safety of children

- ▲ Always bear in mind that Rehadapt products are not toys and should therefore never be used by children or persons with limited judgement capacity as play things, as this could result in the risk of personal injury or the risk of damaging the mounted device or the Rehadapt product. Rehadapt products consist of several individual parts that are assembled to produce a complete device. If the individual parts of the product are removed, the danger of swallowing these removed parts could arise or other risks of accident arising from the now open mounting points. Small children or persons with limited judgement capacity should not be permitted to modify the Rehadapt products unless under the supervision of caregivers.

**2.5 Reporting of serious cases:** Please report all incidents to the following email ([customer-care@rehadapt.com](mailto:customer-care@rehadapt.com)). "Incident" means a malfunction or deterioration of the properties or performance of the Rehadapt product, including use errors due to ergonomic features, as well as any inadequacy in the information provided by Rehadapt or the supplier or any undesirable side-effects. All serious incidents that occur in connection with the product must be reported to Rehadapt.

## 3. Assembly

Rehadapt product must be assembled as indicated in the instructions for use. All general safety instructions and warnings must be complied with. Devices may only be attached to the Rehadapt product in compliance with the specifications of the device manufacturer or in compliance with the specification of Rehadapt. Other supplementary products might be necessary for this, in order to benefit from full use of the product. Improper assembly of Rehadapt Care mounts might impair the safety and operation of the care bed/cart. Please contact the supplier of the care bed/cart if you are in any doubt about the maintenance of the safety and performance ability of the care bed/cart.

## 4. Maintenance and Service

**4.1 Maintenance and Care:** A damaged product or damaged components could injure the user or a third party. Only use a product and components if you are sure they are not faulty. If you are in any doubt, please contact your supplier. Repairs, maintenance and safety inspections may only be carried out by trained and qualified personnel.

### 4.2 Maintenance and service intervals

We recommend yearly maintenance by qualified professionals to ensure the functionality and safety of your Rehadapt products. According to the German Regulation on the Establishment, Operation, and Use of Medical Devices (MPBetreibV) §11, a biennial maintenance is mandatory. Contact your supplier to arrange this.

**4.3 Check of intended use:** In order to ensure intended use all connections on the Rehadapt products must be made in compliance with the instructions for use. Whereby screw connections must not be tightened beyond the maximum permissible torque. Check regularly that safe use is ensured. You can obtain detailed information and help with all questions concerning maintenance and service from your supplier or directly from Rehadapt.

**4.4 General Cleaning Instructions:** All Rehadapt products should be cleaned at regular intervals, in order to ensure flawless hygiene and operation. Cleaning requirements depend on the type of use and usage environment of the product. The user or his/her caregivers bear sole responsibility for the cleaning of a product. Failure to carry out the necessary cleaning could result in loss of the product warranty as well as impairment of the clinical condition and safety of users and/or caregivers. **Attention!** All guidelines listed here are indicative only. More detailed "Cleaning and Disinfection Guidelines" can be found on our website ([www.rehadapt.com](http://www.rehadapt.com)) for each product. We recommend you wear gloves during cleaning. Make sure no water can penetrate the tubes, bearings or joints.

**4.5 Cleaning:** Wipe all parts with a cloth wrung out in warm water and a non-abrasive cleaning agent. Remove any residues with a dry, lint-free cloth. If necessary disinfect

parts paying particular attention to surfaces with which the user and/or caregiver come into contact.

**4.6 Disinfection:** Clean before disinfecting, in order to remove any residues such as dirt. You can then disinfect the surface with 70-percent alcohol wipes. The Rehadapt product can be dried with a paper towel, in order to prevent staining.

**4.7 Service life:** Depending on the intensity of use and the number of times the device is reused, the typically expected service life is 5 years. To achieve this the product must be utilised for its intended purpose and intended use, the specifications in the instructions for use observed and all maintenance and service intervals complied with. The product may be used beyond this period if it is in a safe condition. This typical, theoretical service life is not a guaranteed service life and is subject to a case-by-case assessment by qualified personnel of the supplier as well as its reusability. Using the device after expiry of its specified service life leads to an increase in the residual risks and should only be undertaken after careful and qualified consideration by the supplier. The service life could be shortened depending on the frequency of use, operational environment and care. The typical service life does not apply to wear parts such as wheels and plastic parts that are subject to material-specific ageing and/or wear. This specified service life does not constitute any additional warranty or guarantee.

**4.8 Reuse:** Please contact your supplier in the case you wish to reuse the product or you intend a secondary use. The Rehadapt product may only be passed on for reuse or secondary use if it has been inspected and prepared accordingly in compliance with the safety and performance requirements of Rehadapt.

**4.9 Disposal:** The products from Rehadapt must be disposed of in compliance with the legislation applicable in the respective country.

## 5. Warranty condition, customer service

**5.1 Warranty:** Full 2-year warranty on all Rehadapt products. Rehadapt offers a full warranty for all products of the Rehadapt brand against all types of product defect for a period of two years as of the purchase date, proven by a purchase receipt. This **2-year warranty** is valid for all components delivered with the product and also includes the consequences of wear, improper operation or faulty installation. Rehadapt offers replacement of defective parts or the complete product with new or in mint condition components free of charge within the warranty period. Rehadapt assumes all shipping costs, on-site mounting services are, however, excluded. Damage arising from knowingly using the product for purposes for which it is not intended or willful damage are excluded from the warranty. Rehadapt assumes no liability for damage to other objects that are caused indirectly by defects on Rehadapt products. Rehadapt excludes any liability for compensation for damage to communication aids, medical electric devices, mobility aids or furniture as well as the liability for personal injuries. No claims for compensation can be asserted for any loss of use due to defective Rehadapt products.

Limited 5-year warranty on all Rehadapt products. Rehadapt assumes the warranty for all products of the Rehadapt brand for defects in materials and workmanship for a period of 5 years as of the original purchase date, proven by a purchase receipt. Rehadapt offers at its own discretion replacement of defective parts or the complete product with new or in mint condition components free of charge within the 5-year warranty period, or will repair the original product. Shipping costs are charged to the customer, on-site mounting services are excluded. This limited warranty is valid for all components delivered with the product. This limited warranty only covers problems arising from defects in material and workmanship of the product during its intended use. Any indications of brute force during installation or use will render the warranty void. Wear parts, such as screws, levers and all threads and the consequences of improper use are excluded from the warranty.

**5.2 Customer Service:** For help with technical questions please contact your supplier or Rehadapt at [customercare@rehadapt.com](mailto:customercare@rehadapt.com). You should have your Rehadapt product ready to hand, to enable us to help you as quickly as possible. If you have an Internet connection and the respective terminal devices, please also keep these ready to hand. Please have your invoice number or article number ready to hand. You will find these on your delivery receipt. You can download instructions for use for our products at: [www.rehadapt.com](http://www.rehadapt.com)

## 5.3 In-Country Representatives

**UK** Aventido Limited, Bridge House Ashley Road, Cheshire  
**Responsible Person:** Altrincham WA14 2UT, England, United Kingdom

---

**US** Rehadapt North America LLC, 40 Oser Ave, Suite 40-01,  
**Agent:** Hauppauge, NY 11788 - USA

Rehadapt GmbH, Heinrich-Hertz-Straße 104,  
34123 Kassel – Germany Tel.: +49 561 2207170 [www.rehadapt.com](http://www.rehadapt.com)